



When you use a patient lift, you want the security of knowing that the slings you use are durable and well-made.

At Tollos, we're diligent about you and your patients' safety. So we're careful about ensuring the quality of the sling is up to your standards..

Slings are a Class I Medical Device. Tollos takes compliance with all applicable federal regulations very seriously.

We pride ourselves on our quality and your peace of mind.

---

The Combi Sling is one of the most versatile slings on the market. This sling provides extra support to the hips and mid-back. It is equipped with handles on the back to allow for easy maneuvering of the patient.

The Combi Sling can be used for bed, chair, and floor lifting.

## Combi Sling Features

- Seated sling with head support & padded legs
- Comfortable, yet durable poly-blended fabrics or mesh
- Color-coded binding allows for quick sizing
- Available in washable reusables or in single-patient-use varieties

## General Washing Instructions:

**Note:** Please refer to the enclosed instructions-for-use packaged with all slings.

All Tollos reusable slings should be laundered according to the instructions on the tag that has been affixed to each sling. The following washing instructions are to be followed for optimum performance to preserve and maintain the quality of Tollos reusable slings:

Spot Cleaning: Most disinfection solutions are suitable for spot cleaning the reusable slings & harnesses. Bleach products can be used in a dilution of 100ppm or 0.01%, however, greater concentrations may cause premature deterioration of the fabrics.

Reusable slings should be laundered between patients.

### Washing Instructions:

- Recommended wash cycle: BS 5651:1978 Hospital Laundry Wash (Normal)
- Recommended detergent: ECE Reference Detergent + Sodium Perborate + TAED
- Recommended Wash Cycle: 6 gallons (24 liters) of water at 165°F (74°C) for 15 minutes  
Temp not to exceed 185°F (85°C).
- Recommended Rinse Cycle: 4 cycles at 104°F (40°C) for 3 minutes per cycle
- Recommended Dry Cycle: Drying cycle: BS EN ISO 6330:2012 Procedure E  
Low Temp, not to exceed 120°F (50°C)

Bleach products may be used during the wash cycle in a dilution of 100ppm or 0.01%.

Do not launder single-patient-use slings.

## Sling Guide

Reusable	Reusable Mesh	Single Patient Use	Size	Suitable for:	Binding Color	Safe Working Load
T015CPPF	T015CPPMF	T015CPP-PSS	X-Small	49 - 75 lbs	White	600 lbs/ 272 kg
T015CPF	T015CPMF	T015CP-PSS	Small	75 - 125 lbs	Red	600 lbs/ 272 kg
T015CF	T015CMF	T015C-PSS	Medium	125 - 175 lbs	Yellow	600 lbs/ 272 kg
T015CXF	T015CXMF	T015CX-PSS	Large	174 - 250 lbs	Green	600 lbs/ 272 kg
T015CXXF	T015CXXMF	T015CXX-PSS	X-Large	250 - 400 lbs	Blue	1000 lbs/ 454 kg
T015CXXXF	T015CXXXMF	T015CXXX-PSS	XX-Large	400 - 600 lbs	Orange	1000 lbs/ 454 kg
T015CXXXXF	T015CXXXXMF	T015CXXXX-PSS	XXX-Large	600 - 1000 lbs	Gray	1000 lbs/ 454 kg

Tollos recommends removal of the Combi Sling from under the patient between transfers.

