



When you use a patient lift, you want the security of knowing that the slings you use are durable and well-made.

At Tollos, we're diligent about you and your patients' safety. So we're careful about ensuring the quality of the sling is up to your standards.

Slings are a Class I Medical Device. Tollos takes compliance with all applicable federal regulations very seriously.

We pride ourselves on our quality and your peace of mind.

---

The Hygiene Elite Sling is specially designed for toileting and peri-care. This sling provides extra back support and is comfortable for the patient. It positions the patient precisely, allowing the caregiver easy access for lowering clothing.

The Hygiene Elite Sling is a safe and secure method of transferring a patient.

## Hygiene Elite Sling Features

- Seated sling designed for toileting & peri-care
- Comfortable, yet durable poly-blended fabrics or mesh
- Built-in extra back support for added comfort
- Color-coded binding allows for quick sizing
- Available in washable reusables

## General Washing Instructions:

**Note:** Please refer to the enclosed instructions-for-use packaged with all slings.

All Tollos reusable slings should be laundered according to the instructions on the tag that has been affixed to each sling. The following washing instructions are to be followed for optimum performance to preserve and maintain the quality of Tollos reusable slings:

Spot Cleaning: Most disinfection solutions are suitable for spot cleaning the reusable slings & harnesses. Bleach products can be used in a dilution of 100ppm or 0.01%, however, greater concentrations may cause premature deterioration of the fabrics.

Reusable slings should be laundered between patients.

Washing Instructions:

- Recommended wash cycle: BS 5651:1978 Hospital Laundry Wash (Normal)
- Recommended detergent: ECE Reference Detergent + Sodium Perborate + TAED
- Recommended Wash Cycle: 6 gallons (24 liters) of water at 165°F (74°C) for 15 minutes  
Temp not to exceed 185°F (85°C).
- Recommended Rinse Cycle: 4 cycles at 104°F (40°C) for 3 minutes per cycle
- Recommended Dry Cycle: Drying cycle: BS EN ISO 6330:2012 Procedure E  
Low Temp, not to exceed 120°F (50°C)

Bleach products may be used during the wash cycle in a dilution of 100ppm or 0.01%.

Do not launder single-patient-use slings.

## Sling Guide

Reusable	Size	Suitable for:	Binding Color	Safe Working Load
T015HPPF	X-Small	49 - 75 lbs	White	600 lbs/ 272 kg
T015HPPF	Small	75 - 125 lbs	Red	600 lbs/ 272 kg
T015HF	Medium	125 - 175 lbs	Yellow	600 lbs/ 272 kg
T015HXF	Large	174 - 250 lbs	Green	600 lbs/ 272 kg
T015HXXF	X-Large	250 - 400 lbs	Blue	1000 lbs/ 454 kg
T015HXXXF	XX-Large	400 - 600 lbs	Orange	1000 lbs/ 454 kg

Tollos recommends removal of the Hygiene Elite Sling from under the patient between transfers.

