



When you use patient lift equipment, you want the security of knowing that the slings you use are durable and well-made.

At Tollos, we're diligent about you and your patients' safety. So we're careful about ensuring the quality of the sling is up to your standards.

Slings are a Class I Medical Device. Tollos takes compliance with all applicable federal regulations very seriously.

We pride ourselves on our quality and your peace of mind.

---

Sometimes your patients can support their own weight, they just may need a little help getting up from a seated position. With the Steady Aid Sit-to-Stand device and our own, specially-designed harness, now you can easily assist your patients in standing.

This harnesses, or any sling product, should only be used in the presence of a clinician.

## Steady Aid Harness Features

- Padded harness
- Designed for sit-to-stand lifting
- Comfortable, yet durable poly-blended fabrics or mesh
- Color-coded binding allows for quick sizing
- Available in washable reusables or in single-patient-use varieties



## General Washing Instructions:

**Note:** Please refer to the enclosed instructions-for-use packaged with all slings.

All Tollos reusable slings should be laundered according to the instructions on the tag that has been affixed to each sling. The following washing instructions are to be followed for optimum performance to preserve and maintain the quality of Tollos reusable slings:

Spot Cleaning: Most disinfection solutions are suitable for spot cleaning the reusable slings & harnesses. Bleach products can be used in a dilution of 100ppm or 0.01%, however, greater concentrations may cause premature deterioration of the fabrics.

Reusable slings should be laundered between patients.

### Washing Instructions:

- Recommended wash cycle: BS 5651:1978 Hospital Laundry Wash (Normal)
- Recommended detergent: ECE Reference Detergent + Sodium Perborate + TAED
- Recommended Wash Cycle: 6 gallons (24 liters) of water at 165°F (74°C) for 15 minutes  
Temp not to exceed 185°F (85°C).
- Recommended Rinse Cycle: 4 cycles at 104°F (40°C) for 3 minutes per cycle
- Recommended Dry Cycle: Drying cycle: BS EN ISO 6330:2012 Procedure E  
Low Temp, not to exceed 120°F (50°C)

Bleach products may be used during the wash cycle in a dilution of 100ppm or 0.01%.

Do not launder single-patient-use slings.

## Sling Guide

Reusable	Single Patient Use	Size	Binding Color	Safe Working Load
T016P	T016DP-PSS	Small	Red	350 lbs/ 158 kg
T016	T016D-PSS	Medium	Yellow	350 lbs/ 158 kg
T016X	T016DX-PSS	Large	Green	450 lbs/ 204 kg
T016XX	T016DXX-PSS	X-Large	Blue	450 lbs/ 204 kg
T016XXX	T016DXXX-PSS	XX-Large	Orange	750 lbs/ 340 kg

Tollos recommends removal of the Steady Aid Harness from under the patient between transfers.

