

Ceiling Lifts

Cleaning

Clean the components of the lift, tracks, & rails with a mild, non-abrasive detergent. Wipe exterior surfaces with a damp cloth, ensuring no excess moisture infiltrates the lift or any of its components. Dry with a towel. The interior surfaces of the tracks & rails should be cleaned with a dry duster, such as a Swiffer duster with an extendable handle or similar, at a frequency determined appropriate by the facility, but at least quarterly.

Disinfecting

Disinfect Tollos lifts, IC Belts, spreader bars, tracks and/or rails using a hospital-grade, non-corrosive disinfectant. Follow the disinfectant manufacturer's instructions and facility disinfection protocols. Guard against excess liquid infiltrating the lift housing, spreader bar pivot points, or openings in the tracks/rails.

When cleaning & disinfecting the ceiling lift belt, bring the spreader bar down to the floor, exposing the entire length of the belt. Inspect for damage. With cloth belts look for cuts or fraying. On IC Belts look for cuts, kinks, nicks, or visible steel cabling. Following disinfection, please allow the belt to dry completely before raising the spreader bar.

Tollos Recommends:

- Non-corrosive hydrogen peroxide; i.e. Clorox Hydrogen Peroxide Disinfectant Cleaner, or similar,
- Bleach-based disinfectants in a 1% concentration; i.e. Dispatch Hospital Cleaner Disinfectant with Bleach, or similar,
- Cloth lift belts may be cleaned with a solution of up to 10% hydrogen peroxide, or similar (do not use bleach), or
- Whole room disinfection systems utilizing ultraviolet light or hydrogen peroxide mist.

User Recommendations:

- Ensure that products containing phenols are not used in any concentration.
- Protect slings, harnesses, and cloth lift belts from contact with any bleach solution.

During cleaning & disinfection, inspect the lift & hardware for signs of wear and tear.

Mobile Lifts

Cleaning

Clean the components of the lift with a mild, non-abrasive detergent. Wipe with a damp cloth, ensuring no excess moisture infiltrates the control panel, actuator or spreader bar pivot points. Dry with a towel.

Disinfecting

Disinfect Tollos lifts and spreader bars using a hospital-grade, non-corrosive disinfectant. Follow the disinfectant manufacturer's instructions and facility disinfection protocols. Guard against excess liquid infiltrating the control panel, actuator, and spreader bar pivot points.

Tollos Recommends:

- Non-corrosive hydrogen peroxide; i.e. Clorox Hydrogen Peroxide Disinfectant Cleaner, or similar,
- Bleach-based disinfectants in a 1% concentration; i.e. Dispatch Hospital Cleaner Disinfectant with Bleach, or similar,

(continued)

- Whole room disinfection systems utilizing ultraviolet light or hydrogen peroxide mist.

User Recommendations:

- Ensure that products containing phenols are not used in any concentration.
- Protect slings and harnesses from contact with any bleach solution.

During cleaning & disinfection, inspect the lift & hardware for signs of wear and tear.

Slings & Harnesses

Reusable

Spot Cleaning: Most disinfection solutions are suitable for spot cleaning the reusable slings & harnesses. Bleach products can be used in a dilution of 100ppm or 0.01%, however, greater concentrations may cause premature deterioration of the fabrics.

Reusable slings should be laundered between patients.

Washing Instructions:

- Recommended wash cycle: BS 5651:1978 Hospital Laundry Wash (Normal)
- Recommended detergent: ECE Reference Detergent + Sodium Perborate + TAED
- Recommended Wash Cycle: 6 gallons (24 liters) of water at 165°F (74°C) for 15 minutes
Temp not to exceed 185°F (85°C).
- Recommended Rinse Cycle: 4 cycles at 104°F (40°C) for 3 minutes per cycle
- Recommended Dry Cycle: Drying cycle: BS EN ISO 6330:2012 Procedure E
Low Temp, not to exceed 120°F (50°C)


Bleach products may be used during the wash cycle in a dilution of 100ppm or 0.01%.



Single-Patient-Use

Do not wash: single patient use. Discard sling after use.

Tollos single-patient-use sling labels include a wash indicator. If the sling has been washed and recycled for use again, the wash indicator will turn red. Do not use a sling with a red wash indicator.

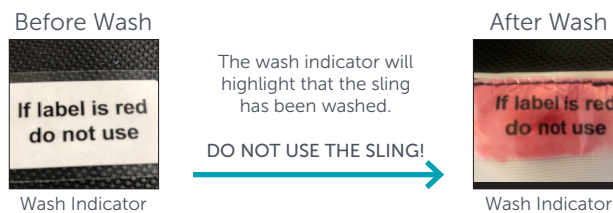

Single-patient-use slings & harnesses may be spot cleaned with disinfecting solutions. Tollos does not recommend the use of bleach in any dilution on single-patient-use slings & harnesses.